


One Step Influenza A&B Test

- A superior high sensitivity of influenza type A and B detection by using monoclonal antibody of influenza virus nucleoprotein (NP).
- Detects a wide range of influenza A subtypes, including H3N2, H7N9, H1N1 (swine flu), H5N1 (avian flu).

Global circulation of influenza viruses

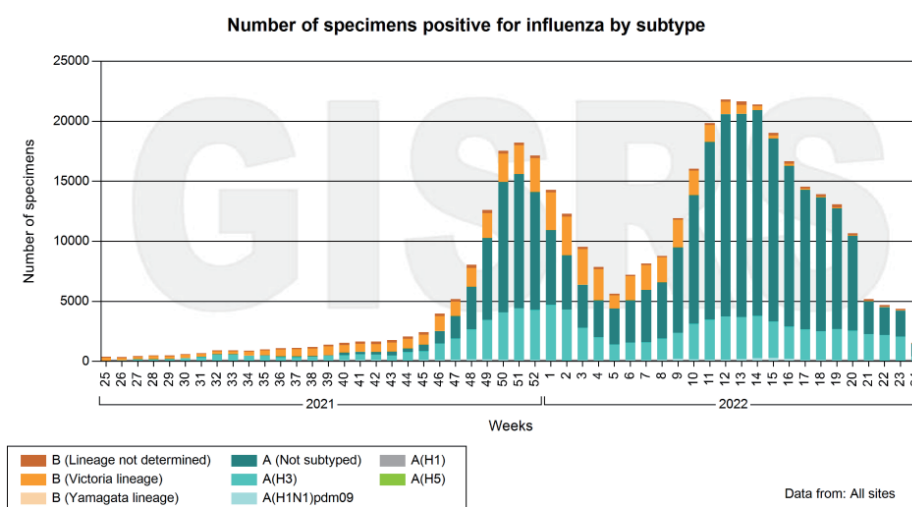

Data source: FluNet (www.who.int/flu-net), GISRS



© World Health Organization 2022

Influenza is a seasonal epidemic respiratory infection with highly contagious risk. The typical symptoms is similar to common cold, but most patients infected with influenza need to be hospitalized. To distinguish influenza from other respiratory infection supports clinical decision making and promotes timely treatment.

- Top influenza rapid test in China with the highest sensitivity.
- Easy-to-use, get reliable result in 15 minutes.
- The test kit contains all necessary materials, no laboratory resources or sample processing is needed.



Test Procedure & Result Interpretation

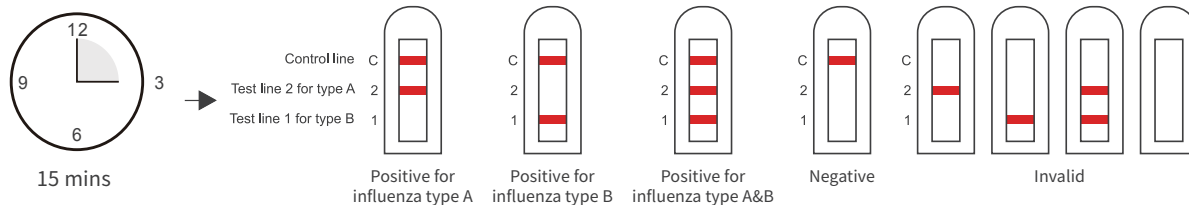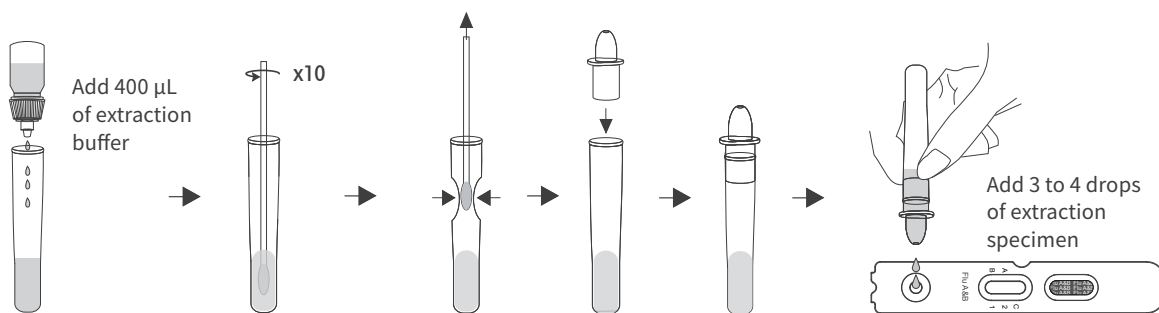

Product Performance (Professional Use)

Wondfo One Step Influenza A&B Test	Sensitivity	Specificity
Influenza A	90.7% (282/311)	95.1% (744/782)
Influenza B	91.5% (270/295)	92.2% (740/803)

Order Information

Product Name	Intended Use	Pack Size	Catalog No.	Qualification
One Step Influenza A&B Test	Professional	20 Tests/kit	W059P0010	CE
	Professional	25 Tests/kit	W059P0011	CE
	Professional	40 Tests/kit	W059P0012	CE
	Professional	25 Tests/kit	W71-C	CE
	Self-testing	1 Test/kit	W71-CH	CE ₀₁₂₃

